


INSTRUCTIONS

CLEAN - LUBE - PURGE - BAG - STERILIZE

For non-aerosol application, add four drops into Drive Air.

TO CLEAN




BIO-C-12

1) External Cleaning: Scrub the exterior of each handpiece with mild detergent and water. Dry.

A1

Internal Cleaning: Holding handpiece in white towel, flush air drive port with Bio Lube Advanced Cleaner using 2-3 second bursts removing water, old oil, and protein build-up. Use your manufacturers' lube adapters to fit quick connect handpieces (**B1**).

A2- Drive Air Port Illustration

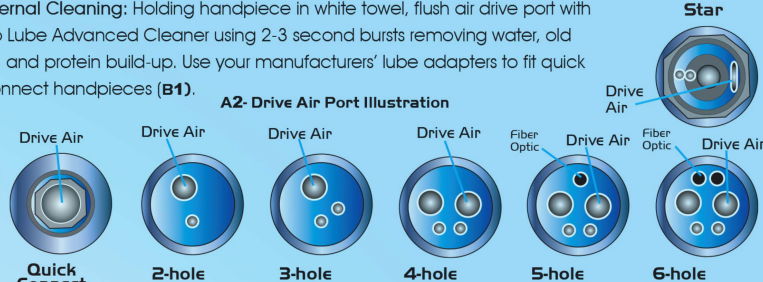

Drive Air

Drive Air

Drive Air

Drive Air

Fiber Optic Drive Air

Fiber Optic Drive Air

Star

Drive Air

Quick Connect

2-hole

3-hole

4-hole

5-hole

6-hole

