

StopLick™

Lick Deterrent Aid



KICK THE LICK HABIT WITH STOPLIK

StopLick Helps Pets Conquer Habitual Licking

Pets, their owners and veterinarians can get fast-acting relief from habitual licking problems with **StopLick™**.

StopLick deters the habitual licking that can cause open sores.

It also discourages pets from removing bandages during healing, allowing the open sores to be more successfully treated.

Designed to discourage pets from licking a sore or wound, **StopLick** is applied over a bandage placed over the site and delivers a mild electric stimulus to the tongue of the pet when licked. Harmless to the pet – the stimulus is similar in strength to one from a watch battery – **StopLick** is activated by moisture. Pets who feel the tingling on their tongue and the unpleasant taste eventually stop licking.

In trials at veterinary clinics, **StopLick** has worked on a high percentage of all sizes and breeds of dogs.

Veterinarians also use **StopLick** to discourage pets from pulling out IVs, Seton drains and urinary catheters or licking surgical stitches on their abdomen and torso.

StopLick **Stops** The Urge to Lick

- ✓ Unique, simple
- ✓ No drugs
- ✓ Highly effective, fast-acting relief for most pets
- ✓ Accepted by pets and owners
- ✓ Field-tested
- ✓ Non-hazardous
- ✓ Alternative to Elizabethan Collar, Bitter Apple Spray
- ✓ Color-coded

EFFECTIVE TREATMENT FOR:

- ✓ Lick Granuloma
- ✓ Licking, tearing at surgical stitches
- ✓ Pulling out IVs, Seton drains, urinary catheters
- ✓ Dogs, cats or other small animals
- ✓ Deterring other pets from tampering with bandages

StopLick is offered in color-coded 1.62 and 3 inch widths. Each easy-to-use dispenser holds 12 strips. Each lick deterrent aid is individually packaged with directions for application.

Size	Width
Wide	3 inch
Narrow	1.62 inch

StopLick™

Lick Deterrent Aid

Kick The Lick Habit

Applying StopLick is Simple

To activate StopLick:

Remove brown paper backing from the front of the strip on the battery end.

Fold end over, so the B symbol is placed onto the top of the battery. The target holes should line up to each other. Press material firmly onto and around the battery to ensure activation.

For leg applications:

Remove the white paper backing from the StopLick.

Apply StopLick over the bandage starting with the battery end, wrapping around the bandage and back over itself, just past the battery, allowing a 1 inch overlap. Trim off excess material with scissors so the breathe holes are not sealed off.

For other than leg applications:

Before applying over the bandage, fold battery end back onto the adhesive on the backside of StopLick. Now StopLick is ready to be applied to the bandage area.


StopLick can be cut to length from the opposite end of the battery. Trimming does not change the effectiveness of the strip.

For post-surgical use:

StopLick can be applied to discourage pets from pulling out IVs, Seton drains and urinary catheters or licking surgical stitches on their abdomen and torso.

Visit www.StopLick.com to view an educational video on StopLick application.

Dr. Joshua W. Stewart,
Wales Animal Clinic, WI
and StopLick patient.

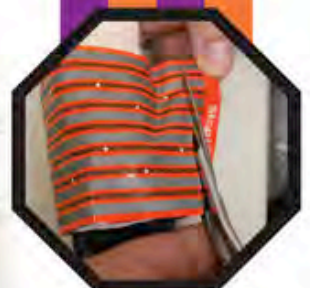

Kick The Lick Habit

MAI Animal Health - UNITED STATES
605 Pro-Ject Drive
Elmwood, WI 54740
1.800.447.0687
sales@maianimalhealth.com

MAI Animal Health - EUROPE
4 Merlin Way
Bowerhill Industrial Estate
Melksham, Wiltshire SN12 6TJ
+44 (0) 1225 791 184
sales.uk@maianimalhealth.com

MAI Animal Health - AUSTRALIA
5-7 Burrundulla Road
P.O. Box 1147
Mudgee NSW 2850
+61 (0) 2 6372 6622
sales.au@maianimalhealth.com

MAI Animal Health - NEW ZEALAND
3A Marken Place
Glenfield, Auckland 0627
+64 (0) 9 444 2115
sales.nz@maianimalhealth.com







Let the healing begin


Not only do your clients trust you to make the right decisions to keep their pets safe during surgery, their trust extends beyond the initial surgery or injury. **LikShield™** is a wound management product designed to prevent post-surgical wound licking by pets that in many cases can prolong the healing process and lead to infection.

LikShield, with its FDA-approved adhesive, can be applied directly to the skin around an incision or over a bandaged area. **LikShield** is available in a resizable 9" x 6" patch for wide-area coverage. **LikShield** provides a safe, mild electric stimulus to the pet's tongue when licked to deter the pet from licking.

The tingling sensation and an unpleasant taste will prevent pets from licking, allowing their wound to heal.

LikShield can be safely used for:

-  Surgical incisions
-  Wound management
-  Splints and casted areas
-  Urinary catheters
-  Seton drains
-  IVs

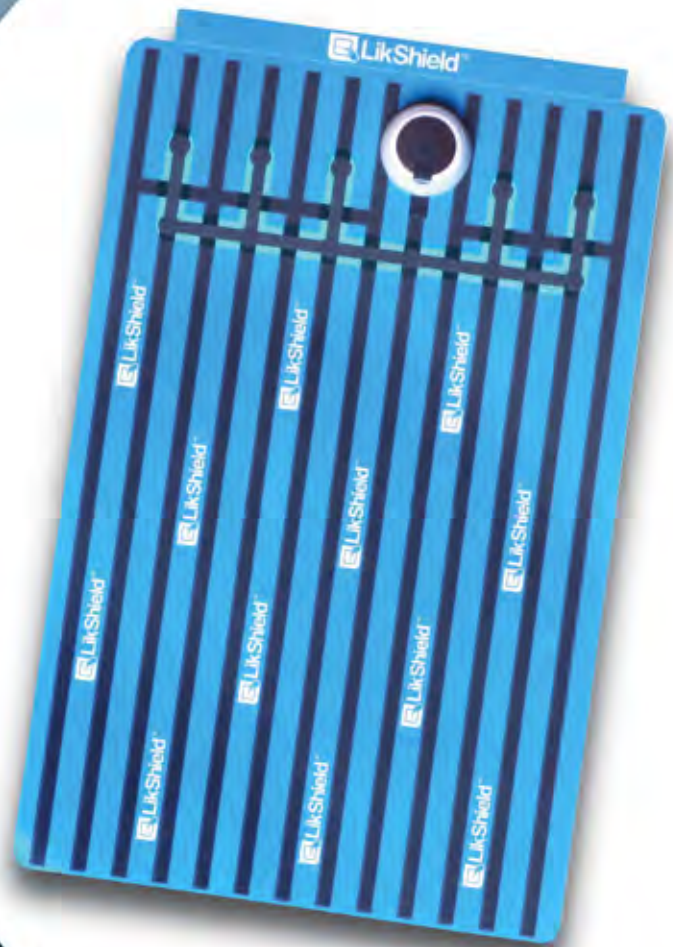





BUSINESS PARTNER

ARO
SYSTEMS 



P: 1300 596 664 www.arosystems.com.au
 +61 2 6059 6664 info@arosystems.com.au
 3/67 Wigg Street, Wodonga 3690 Victoria

-  The resizable 9" x 6" patch can be fitted to dogs, cats and other companion animals.
-  LikShield is ideal for wide-area coverage: abdomen, loin, back, tail head, etc.
-  Safe, field-tested and non-hazardous to pets and their owners.
-  LikShield is highly effective and fast acting. A battery retention ring keeps the non-hazardous battery secure.








Surgery is a stressful situation for most pets. LikShield is an easy, safe and low-maintenance product to prevent the natural licking instinct, allowing surgical wounds to heal. Deterrent sprays and Elizabethan Collars can be uncomfortable for pets, unlike LikShield which does not obstruct movement and is applied one time for a consistent response. Allowing pets more freedom can help the healing process.



LikShield is flexible in design to function wherever post-surgery or post-injury licking occurs. With just a few simple steps, you will have your patient on its way to a lick-free recovery.

In veterinary trials, LikShield technology has proven to be over 90% effective, regardless of size, age or breed of pet.


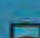

Simple application with reliable results.

To apply LikShield:

-  Make sure the area where LikShield is to be applied is clean and oil-free. Clean the application area with rubbing alcohol.
-  Cut LikShield to size if needed; cutting LikShield will not change its effectiveness.
-  Remove paper backing from LikShield. Apply LikShield directly over a bandaged incision.
-  Trim LikShield to about a 1-inch overlap, covering the battery if wrapping the LikShield around a limb.
-  Keep area dry.

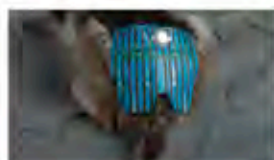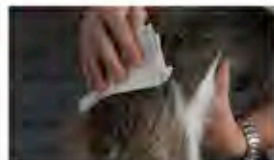

To remove LikShield:

-  Warm LikShield with a warm washcloth to aid in the removal of the product.
-  Discard the non-hazardous patch by normal means of disposal.
-  LikShield should only be applied by veterinarians and trained veterinary technicians.



Visit www.LikShield.com for more information on LikShield technology.