Estimado cliente

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EN

Dear customer,

Thank you for choosing a quality product from Dentaureum. Please read these instructions for use to ensure you use this product in a safe and simple way and that you and your patients gain full benefit. In case of questions or ideas, please contact your local representative.

1. Manufacturer

Dentaureum GmbH & Co. KG
Turnstr. 31 | 75228 Ispringen | Germany

2. General device description

Variety click/Variety click SP screws are special universal expansion screws with a screw locking feature (Fig. 1) which prevents unwanted reversing and can be used in the lower jaw to create space and prevent tipping of the anterior and posterior teeth as well as to open and close gaps in both the upper and the lower jaw. They can also be used as distraction screws, or for early palatal expansion of the upper jaw at the beginning of the mixed dentition phase. The Variety click/Variety click SP expansion screws are welded or soldered to preformed bands or cap splints using the retention legs. The appliances are cemented as usual. Follow the instructions from the cement manufacturer.

3. Intended purpose

Distalization, mesialization or gap closure by movement of a single tooth and early palatal expansion in the upper jaw.

4. Intended users

All products are intended for dental professionals only.

5. Indications

In case of transverse under-development of the upper jaw. In case of transverse under-development of the lower jaw. In rarer cases, also for sagittal tooth movements.

6. Contraindications

Palatal expansion in patients whose second mixed dentition phase is completed. Stainless steel expansion screws must not be used in patients with a nickel and/or chromium allergy. Warnings are found on the respective product labels. Also see 9. Composition.

7. Patient target group

All patients for whom a fixed appliance with an expansion screw can be placed as part of orthodontic treatment.

8.1. Fabricating an appliance with a Variety click/Variety click SP expansion screw

Preformed bands or cast cap splints are fitted on the lower jaw first premolars and the first molars, each left and right. The bands/cap splints are adapted on a hard plaster model. The retention legs of the Variety click/Variety click SP expansion screw are bent so that they can be welded onto the adapted bands/cap splints (Fig. 3). The Variety click/Variety click SP expansion screw should "float" freely 1 mm – 2 mm above the gingiva (Fig. 4). To avoid any leverage effect, ensure that the direction of force runs parallel to the occlusal plane.

8.2. Tips for working with Variety click/Variety click SP expansion screws

In order to bend the retention legs gently and easily, we recommend using the universal bending key (REF 611-122-00, Figs. 5 + 9) without the aid of additional pliers (lever principle). If the universal bending key is not available, the retention legs must be stabilized during bending by holding the expansion screw body with a pair of sturdy pliers, e.g. wire bending pliers Maxi, Premium Line (REF 013-522-00), in order to prevent the retention legs from fracturing (Fig. 6).

9. Composition

The Variety click/Variety click SP screws are made of the following materials: DIN 1.4301, DIN 1.4305 and DIN 1.4310. The material composition is included in the orthodontics materials list; please refer to the orthodontics catalog or [www.dentaureum.com](https://ec.europa.eu/tools/eudamed). The SSCP is available at <https://ec.europa.eu/tools/eudamed> and [www.dentaureum.com](https://ec.europa.eu/tools/eudamed).

10. Safety information

Always use the blue safety key supplied to prevent swallowing. The product should not be used if there is a known allergic reaction to one or more of the material components. If the packaging shows serious damage, the product must be checked for integrity and cleanliness before use, and if necessary must be disposed of.

11. Information for single use products

The product is intended for single use only. Reconditioning (recycling) of the product and its reuse is not permitted, as perfect functionality of the product cannot be guaranteed.

12. Further information

Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the user and/or the patient is resident are informed accordingly.

13. Product range

For the complete range of Variety click/Variety click SP expansion screws, please see our orthodontics catalog or visit [www.dentaureum.com](https://ec.europa.eu/tools/eudamed).

14. Quality information

Dentaureum ensures faultless quality of its products. These recommendations are based upon Dentaureum's own experiences. The dental professional is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as Dentaureum has no influence on the processing on site.

15. Explanation of symbols used on the label

Please refer to the label. These Instructions for use and additional information can be found at [www.dentaureum.com](https://ec.europa.eu/tools/eudamed) (Explanation of symbols REF 989-313-00).

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The product should not be used if there is a known allergic reaction to one or more of the material components. Should the dental professional and/or the patient become aware of serious problems arising from the use of the product, it is important that the manufacturer and the competent authority in the country in which the dental professional and/or the patient is resident are informed accordingly.

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