3D Isoliermittel
3D separating medium
Isolant 3D
Barniz aislante 3D
Isolante 3D

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Dear customer,

Thank you for choosing a quality product from Dentaaurum. It is essential to read these instructions carefully and adhere to them to ensure safe, efficient use and ensure that you and your patients gain full benefit. Instructions for use cannot describe every eventuality and possible application.

In case of questions or ideas, please contact your local representative.

As our products are regularly upgraded, we recommend that you always carefully read the current Instructions for use supplied with the product and stored in the internet at www.dentaaurum.com, even though you may frequently use the same product.

1. Manufacturer

Dentaaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. Product description

The 3D separating medium forms a separating layer between the 3D-printed plastic model and Orthocryl®/Orthocryl® LC.

To fabricate dental models, we recommend using high quality plastic models that were made using DLP (Digital Light Processing). e. g. prime4me® base model (REF 430-110-10). Besides being highly accurate and therefore ensuring very precise fitting, these models allow for the usual process in the laboratory.


Due to the large variety of materials for model printing, it is essential to test the suitability of the 3D separating medium before its use.

3. Processing instructions


3.1 If necessary, block out undercuts in the area of the teeth/jaw during fabrication using the 3D printing process.

3.2 Scrape the plastic model and, if required, create holes/slits for the expansion screw.

3.3 Place the expansion screw in position and bend the wire elements.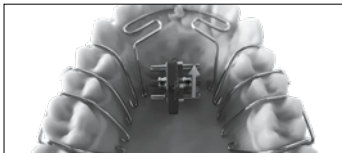

3.4 Apply the 3D separating medium evenly with a brush over the entire model before the expansion screw (or screws) and wire elements are finally fixed in place and allow the separating medium to dry thoroughly. The separating medium is completely dry when it no longer feels sticky. Repeat the process in order to achieve an even separating layer everywhere.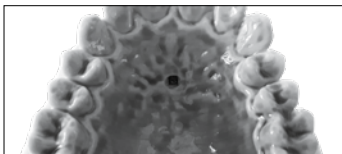

In order to accelerate the drying of the separating medium or to shorten the drying phases, careful blowing with compressed air or a household hair dryer can be helpful. For faster drying of the separating medium, the model can also be placed in a light curing unit for a few minutes*.


* The time required in the light curing unit may vary depending on the size of the workpiece and the amount of separating medium. The drying time also depends on the type of device – with the Solidilite V unit from Shofu, up to 3 minutes should be enough.

3.5 If required, block out undercuts in the tooth/jaw area and fix the expansion screw(s) and wire elements as usual on the dried plastic model. Use Thermo wax (REF 120-170-00) to avoid a milky effect in the plastic.


3.6 If necessary, use a wax strip as a boundary for the plastic.

3.7 Process the plastic as usual. Please follow the corresponding instructions for use.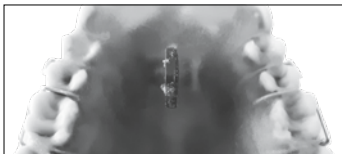

3.8 Residues from the separating medium can easily be removed completely with a steam cleaner or with water and a brush.

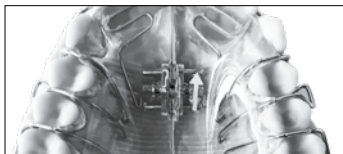4. Storage and shelf life

Protect the 3D separating medium from contamination and store in a closed container. Do not use after the expiry date.

5. Availability

3D separating medium, 100 ml, REF 162-801-00

6. Quality information

Dentaurum ensures faultless quality of its products. These recommendations are based upon Dentaurum's own experiences. The user is solely responsible for the processing of the products. Responsibility for failures cannot be taken, as we, Dentaurum, have no influence on the processing on site.

7. Explanation of symbols used on the label

Please refer to the label. Additional information can be found at www.dentaurum.com (Explanation of symbols REF 989-313-00)

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Notes

This image shows a full page of a handwriting practice worksheet. It consists of approximately 20 horizontal rows. Each row is defined by two parallel dashed lines, creating a series of uniform gaps for letter height. The lines are evenly spaced across the entire page, providing a guide for consistent letter formation. There is no text or other markings on the page.

Notes

[illegible]

Dentaurum

Germany | Benelux | España | France | Italia | Switzerland | Australia | Canada | USA
and in more than 130 countries worldwide.

DENTAURUM
QUALITY
WORLDWIDE
UNIQUE

- Informationen zu Produkten finden Sie unter www.dentaurum.com
- For more information on our products, please visit www.dentaurum.com
- Vous trouverez toutes les informations sur nos produits sur www.dentaurum.com
- Descubra nuestros productos en www.dentaurum.com
- Informazioni su prodotti sono disponibili nel sito www.dentaurum.com

Stand der Information | Date of information | Mise à jour
Fecha de la información | Data dell'informazione: 09/21

Änderungen vorbehalten | Subject to modifications | Sous réserve de modifications
Reservado el derecho de modificación | Con riserva di apportare modifiche

D
DENTAURUM