



Innoscreen
"Innovation changes the world"

Drugs of Abuse Quick Test

- High quality
- Safe and trusted
- Cost-effective
- Ease of use
- Australian standards

Results in 5 minutes



Screens for 6 Drugs, Alcohol, & 6 Adulterants.

Includes:

- ◆ Temperature strip
- ◆ Security seals

Drugs of Abuse test choices

TEST	CALIBRATOR	CUT-OFF (ng/mL)
AMP	d-Amphetamine	300
BZO	Oxazepam	200
COC	Cocaine	300
MOR	Morphine	300
MET	Methamphetamine	300
OPI/MOR	Morphine	2000
THC	11-nor- 9-THC-9-carboxylic acid	50
ALC	Alcohol	0.02%

ADULTERATION:

- ◆ Oxidants
- ◆ Specific Gravity
- ◆ PH
- ◆ Nitrites
- ◆ Glutaraldehyde
- ◆ Creatinine


Done in 6 easy steps:

- 1 Collect the specimen. Sample should meet minimum level as indicated on the cup.
- 2 Apply lid back onto cup, ensuring it is screwed on tightly.
- 3 Donor dates and initials the security seal label.
- 4 Operator checks the cap for tightness and attaches the security seal label over the cap.
- 5 Remove the peel off label. Check the temperature strip label at 2-4 minutes after specimen collection.
- 6 Results are ready to read at 5 minutes. Do not interpret results after 10 minutes.


QUICK FACTS

INTERPRETATION OF RESULTS



POSITIVE
only one coloured band appears, in the control region (C).



NEGATIVE
Two coloured bands appear on the membrane. One in the control region (C), and another in the test region (T).



INVALID
Control band fails to appear. Results from any test which has not produced a control band at the specified time must be discarded.

+ Drug positive tests will not generate a coloured line in the specific test line region of the strip because of drug competition.

— A drug negative urine specimen will generate a line in the test line region because of the absence of drug competition.

✓ The DOA tests are stable and ready to use, with an outstanding performance.



Icon Medical Supplies Pty Ltd
Unit B, 75-77 St Hilliers Road Auburn NSW 2144
Tel: 1300 49 99 99
www.iconmedical.com.au
e: sales@iconmedical.com.au