



Instructions for use

DESCRIPTION

Cavex Bite&White ExSense is a water-based gel with mint aroma that contains hydroxyapatite and potassium nitrate. Cavex Bite&White ExSense controls sensitivity and enhances whiteness of the teeth by hydroxyapatite remineralization.

PACKAGING

Tube 42 ml – 50 gram (sufficient for approximately 20 treatments)

COMPOSITION

Key ingredients:

Hydroxyapatite 2%, Potassium nitrate 0,1%

INDICATIONS FOR USE

- After in-office and/or at-home tooth whitening
- After professional tooth cleaning
- Prevention and control of sensitivity

CONTRA-INDICATIONS

Do not use Cavex Bite&White ExSense in cases of allergies to mint aroma or any of the ingredients.

HANDLING PROCEDURE

- Prepare and clean the teeth using brush and floss.

APPLICATION:

WITHOUT TRAY:

Wash hands thoroughly with soap and water. Dispense a small amount of Cavex Bite&White ExSense onto a clean finger. Gently apply gel on all surfaces of the upper and lower teeth. Allow gel to remain on tooth for 10 minutes. Spit out any excess gel and rinse mouth with water.

WITH TRAY:

Dispense a small ribbon of Cavex Bite&White ExSense across the entire tray. Insert tray and gently rotate tray left and right to assure gel is distributed across teeth. Gently bite down on the tray and leave gel in place for 10 minutes. Remove tray, spit out any excess gel and rinse mouth with water. Rinse tray with cold water.

- Always close the tube with the cap immediately after use.
- For best results, do not drink or eat for at least 30 minutes after the application.
- Use product every day for 10 days or as instructed by your dental professional.

WARNINGS & FIRST AID INSTRUCTIONS

In case of contact with the eyes, flush thoroughly with plenty of water and seek medical attention.

Do not use the material after the expiry date.

STORAGE ADVICE

Store in a cool place, dry place (frost-free) and protected from direct sunlight.

*Your impression is our concern
Made in The Netherlands*

Our technical advice, whether verbal, in writing or by way of trials, is given in good faith but without warranty, and this also applies where proprietary rights of third parties are involved. It does not release you from the obligation to test the products supplied by us as to their suitability for the intended processes and uses.

The application, use and processing of the products are beyond our control and, therefore, entirely your own responsibility. Should, in spite of this liability be established for any damage, it will be limited to the value of the goods delivered by us and used by you. We will, of course, provide consistent quality of our products within the scope of our General Conditions of Sale and Delivery.