

Cavex ImpreSafe disinfection protocol

Bactericide, fungicide, virucide disinfectant suitable for disinfecting alginate, polyether and silicone impressions

Mixing ratio: 30ml. disinfectant on 1 of liter water

Exposure time: 3 minutes



① Squeeze bottle and fill measuring cup up to 30ml.



⑤ Lower impression into fluid. Disinfect for 3 minutes by using the timer.



② Pour water in reservoir first, (see TIP1) up to indicated stripe (app. 1 liter).



⑥ After 3 minutes, take the tray out of disinfectant fluid.



③ Pour disinfectant into reservoir.



⑦ Rinse of with water.



④ Place impression on tray in container.



⑧ Shake of excess water and put impression in a plastic bag. Seal airtight. Mark the bag as "disinfected".

TIP1: When filling the container, always put the water in first and then the disinfection fluid. This prevents foaming.

TIP2: Refresh the fluid every week or when the fluid looks contaminated.