

Palajet®/ Palajet® duoflask

The pneumatic injection unit for full dentures.

Our injection unit is not only suited to match Pala denture acrylic's material characteristics exactly, but is also very easy to use. The ingenious feed system cuts out dosage errors and the working characteristics are always identical.

Injecting:

The piston presses the premixed acrylic dough into the flask. The injection moves from bottom to top: This allows even, bubble free filling. Once the lever has been activated, the injection process begins automatically. At cold curing materials the flask remains a further 5 minutes in the injection unit before final polymerisation takes place.

The advantages in summary:

- No increase in vertical dimension
- High precision and excellent fit
- Hardly any finishing needed
- Easy to operate
- Maintenance-free
- Effortless operation

For our Dental Science informations about Palajet please take a look at: kulzer.com/palajet



Palajet duoflask: Preparing and polymerising two dentures in one go.