



a.max AM-I

Optimum Capsule Mixer



- ◆ User friendly
- ◆ Ergonomic design
- ◆ Maintenance free
- ◆ High Technology Performance
- ◆ Controlled, consistent mix
- ◆ Electrical safety standards met
- ◆ Noiseless and stable operation
- ◆ Easy operation and sturdy capsule holder

a.max^{AM-I} Capsule Mixer


High Performance

a.max capsule mixer is a new mixing unit for all capsules including amalgam capsules. It can blend silver amalgam capsules and spills of dental. It has safe operation, stable, extremely quiet, suitable structure and symmetric blend.

a.max high-speed amalgamator is controlled by a microprocessor, which ensures a precise, controllable and consistent mixing. With a safety cover, **a.max** mixer will deactivate automatically when you open the front cover any time to access the capsule. The molded capsule holder is easy for capsule loading & unloading. **It revolves according to "8" model, with the width of the oscillating range, the mixing frequency, and your selected time (1-30 seconds).** A stylish compact design is easy to clean and maintain. The best choice of durable, reliable and affordable unit in the market.

Easy to use

- > Just turn on the Main Power Switch, it shows **Mode-1** with 10 seconds.
- > Adjust the "+" or "-" buttons to the desired time in seconds. Timer can be set from 1~30 seconds within 2 memory modes.
- > After putting amalgam capsule, close the cover and push the **"START/STOP"** button to start the operation.

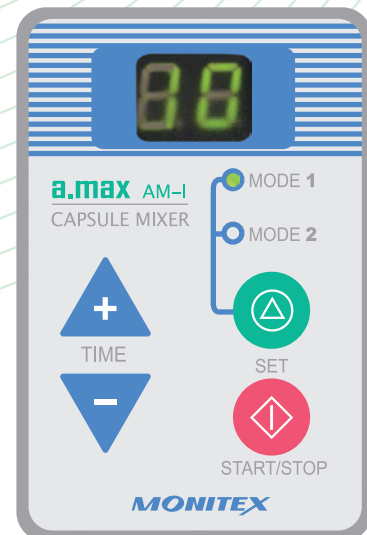

Specification

- I/P: DC 24V 1.5A 36W
- Adapter: Input AC100 ~ 240V 50/60Hz
Output DC 24V
Current 1.5A
- Setting Timer: 1 ~ 30 seconds
- Mode: 2 memory Modes
- Operation Temperature: 10 °C ~ 40°C
- Dimension: 210mm X 200mm X 150mm
- Weight: 2.9kg



6F, 70 GUANG-FU ROAD SEC.1, SAN-CHONG CITY, TAIPEI, TAIWAN.
TEL : 886-2-29993869 FAX: 886-2-29993647 E-mail : monitex@fomed.com.tw
<http://www.monitex.com.tw>



2009.06