marathon


Mutti600

MANUAL

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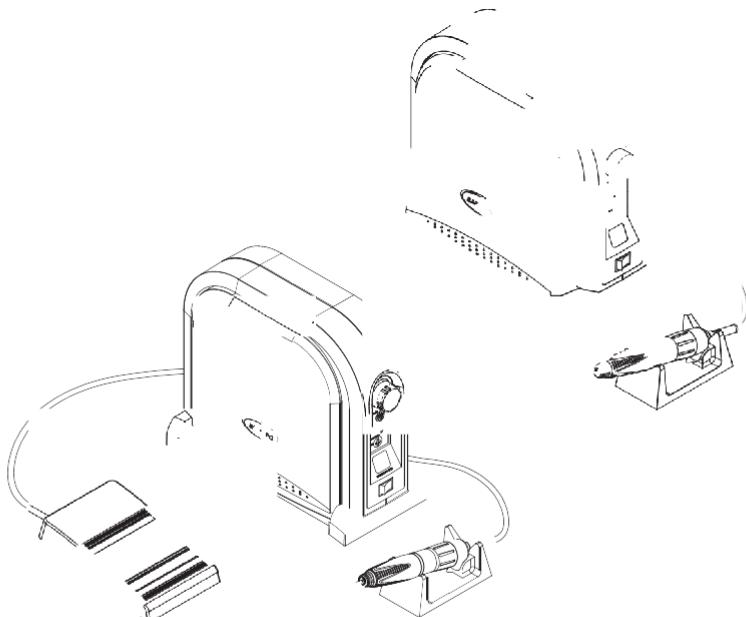
saeyang

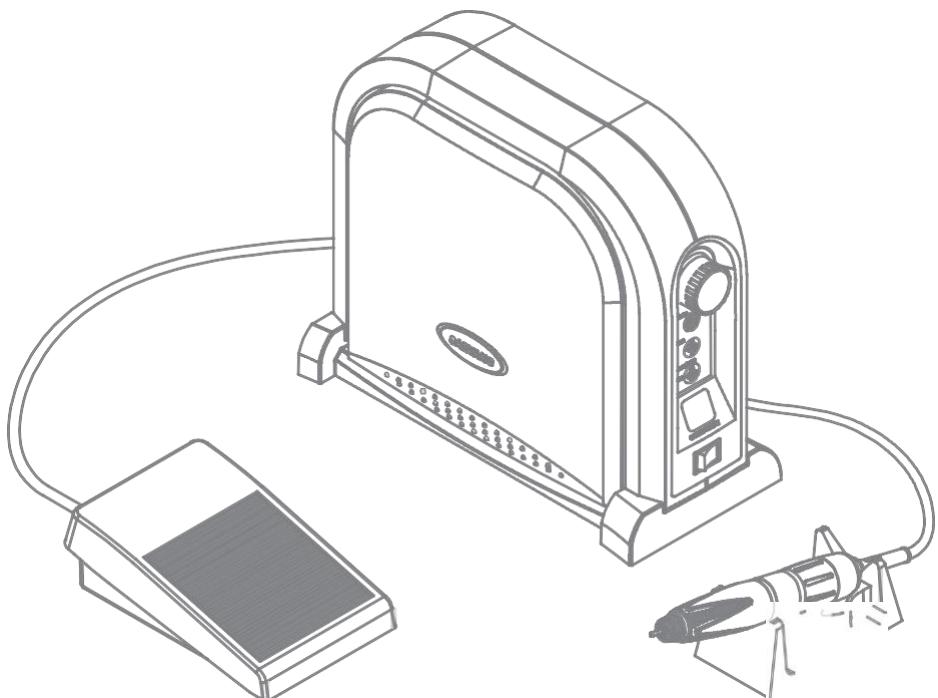GBI Read this User's Manual thoroughly for safety before preparing the unit for use.

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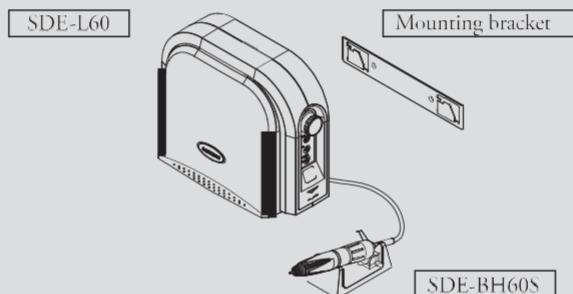
Fig-3

Contents

1. Components
2. Electric power
3. Installation
4. Mounting & electric connection
5. Components
6. What to do before operation
7. How to operate
8. Care and maintenance
9. Technical data

1. Complete set

Make sure that the following items(1-1.3) are included in your unit package:

1.1 Desktop model(SDE-S60)[Fig. 1]

- Control unit stand(SDE-S60/L60)
- Control unit stand
- Motor handpiece(SDE-BH60S)
- Motor handpiece stand
- Foot pedal(SDE-FS60)

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
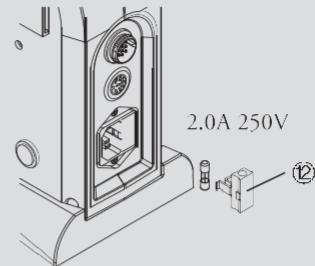
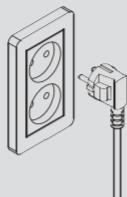
Fig-5

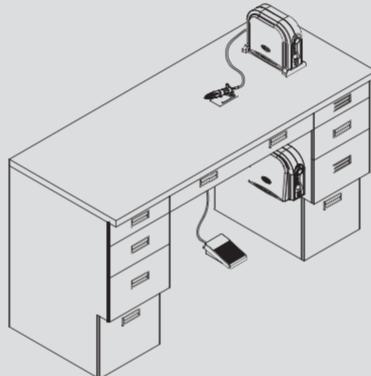2. Electric power

- 2.1 Check the electric rating label (AC200-240V, 50/60Hz) affixed to the rear of the unit .
- 2.2 Use only the fuse with allowable electric voltage(2.0A 250V) for fuse holder @.

CAUTION

1. Never plug or unplug the power cord with wet hands to avoid electric shock.

3. Installation

- 3.1 The SDE-S60 is a desktop model.
- 3.2 The SDE-L60 is a knee control model.

CAUTION

1. The control unit should rest on a flat, steady surface.
2. Use the unit in locations that have indoor temperature(0- 40°C). Avoid using the unit in an excessively dusty, warm or humid location, due to electric shock or damage to the unit.

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
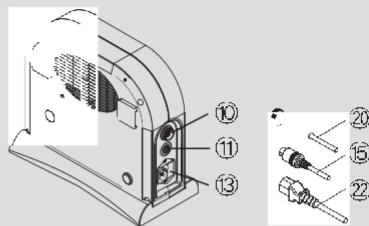
Fig-8

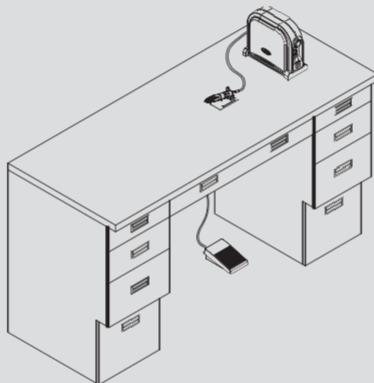4. Mounting and electric connection

- 4.1 Desktop model(SDE-S60)
- 4.1.1 Tighten the bolts for control unit stand @) on the bottom of control unit CD while keeping knee control panel @ pressed fully[Fig. 6].
- 4.1.2 Plug moror cord into motor connector @ located in the rear of the control unit and then righten plug nurs[Fig. 7].
- 4.1.3 Plug foot pedal cord @) into footpedal connector @ located in the rear of the control unir[Fig. 7].
- 4.1.4 Plug power cord @ into powerconnector @ in the rear of the control unir[Fig. 7].

CAUTION

- I. After setting up the control unit stand, make sure that the knee control pedal is fully closed.
2. Make sure that the bolts for the control unit stand are firmly tightened to set the control unit in proper condition.
3. Make sure that all the cords are connected safely and properly before plugging the power cord.

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
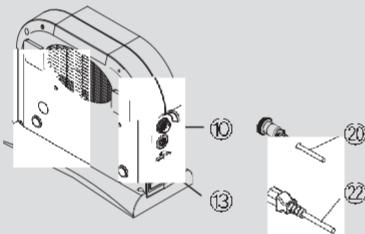
Fig-11

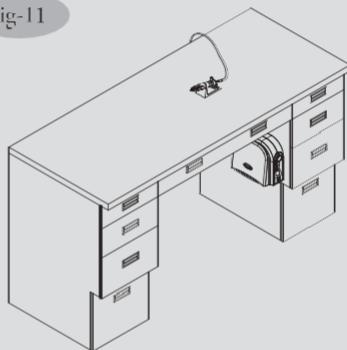4.2 Knee control model(SDE-L60)

4.2.1 Set mounting bracket ⑮ to a proper place for knee control under your technical desk as shown in Fig. 9, using bolts and screws.

* The recommended height of the mounting bracket from the bottom is:

- 1) 520~550mm for oriental people
- 2) 530~580mm for western people

4.2.2 As shown in Fig. 9, firmly mount the control unit bracket in the proper direction after checking their holes.

4.2.3 Plug motor cord ⑯ into motor connector ⑰ in the rear of the control unit in the proper direction, and then tighten the plug nuts firmly[Fig. 10].

4.2.4 Plug power cord ⑯ into power connector ⑰ located in the rear of the control unit[Fig. 10].

CAUTION

1. While setting the mounting bracket, special care should be taken for a bolt or screw head not to be out of the surface of the bracket.

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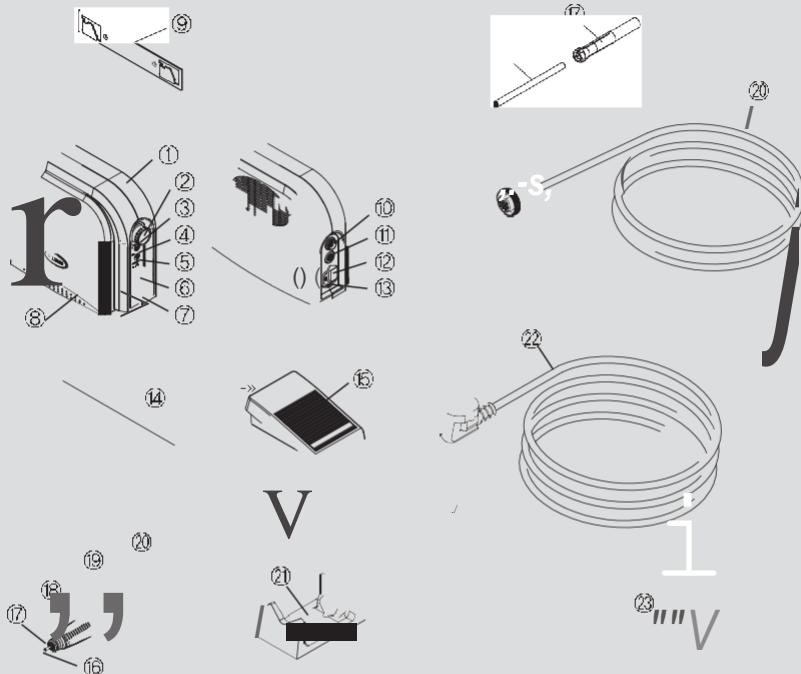
@ Fuse holder
 @ Power connector
 @ Control unit stand
 @ Foot pedal
 @ Test bur
 @ Chuck
 ® Chuck handle
 I @ Motor
 @ Motor cord
 @ Handpiece stand
 @ Powercord
 @ Tools

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
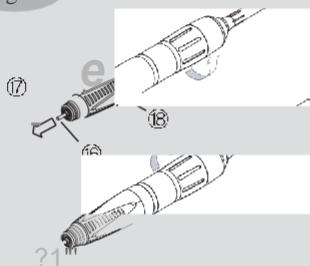
Fig-15

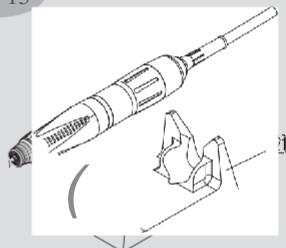6. What to do before operation

- 6.1 Set power switch (J) to "O," OFF position, whenever connecting the motor handpiece and foot pedal [Fig. 13].
- 6.2 The motor speed can be controlled by speed control knob (2) within the limit of 0-40,000RPM [Fig. 13].
- 6.3 The operation of turbo switch (⑧) is possible only at the motor speed of 40,000RPM, therefore, which is accelerated to 50,000RPM [Fig. 13].
- 6.4 To control the motor speed by means of knee control panel (⑨) and foot pedal (⑩), it is required to press run switch (5) to start the motor. Beeps are made by pressing auto cruise switch (5) for 2 seconds and then the motor stops running. The motor speed is variable by the pressed volume of knee control panel (⑨) and foot pedal (⑩) [Fig. 13].

- 6.5 When knee control panel (⑨) and foot pedal remain at the same position for 2 seconds, the mode is changed to auto cruise after the beep. After that, the motor keeps running at the uniform speed even after releasing knee control panel (⑨) and foot pedal (⑩). The auto cruise mode can be released by pressing either knee control panel (⑨) or foot pedal (⑩) again, or by pushing auto cruise switch (5) [Fig. 13].
- 6.6 The motor speed can be controlled by knee control panel (⑨) and foot pedal (⑩) within the limit set by speed control knob (2).
- 6.7 When the LED of forward/reverse(Rev) selector switch (④) is off, the motor is running counterclockwise [Fig. 13].
- 6.8 Test bur can be replaced by turning chuck handle clockwise. After replacing the test bur or a polisher hing point, the chuck handle must be turned counterclockwise to be securely held [Fig. 14].

CAUTION

1. To use the turbo switch for 40,000RPM or more, it is required to use the recommended bur only.
2. Always remove dirt before replacing a test bur/polishing point.
3. After replacing a test bur/polishing point, it is necessary to make sure that the chuck is securely held before running the motor.
4. While the motor is running, **DO NOT** try to

adjust the chuck handle to prevent the unit from being damaged.

5. When the motor handpiece is not in use, it is recommended for a test bur/ polishing point to be kept inserted into the motor handpiece.
6. Always put the motor handpiece on its stand while in use. Special care must be taken not to drop it on the floor.

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
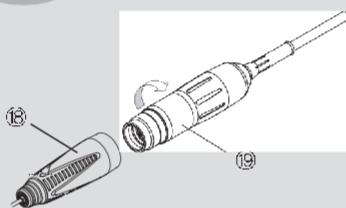
Fig-21

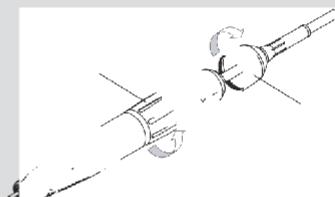8. Care and maintenance

- 8.1 How to mount and dismount the collet chuck
 - 8.1.1 Turn the collet chuck counterclockwise while opened to dismount. If the collet chuck can not be turned manually, use tools to dismount it[Fig. 19].
 - 8.1.2 To mount the collet chuck, it is required to first open the chuck handle and then insert the collet chuck and rest bur. Then, fully turn the collet chuck clockwise with fingers and then turn a fifth counterclockwise again to make the rest released.
- 8.2 How to disassemble the chuck handle and motor
 - 8.2.1 The chuck handle and motor are disassembled by continuously turning the chuck handle counterclockwise[Fig. 20].

8.3 How to disassemble the motor and its cord

- 8.3.1 The motor cord is disassembled by continuously turning the motor counter-clockwise[Fig. 21].

CAUTION

1. Thoroughly clean the collet chuck and rest bur before reassembled. It is recommended to clean them once a week, for safety.
2. Special care must be taken for the chuck handle and motor to be free from and dirt during disassembly.
3. Only the skilled engineer is authorized to disassemble the spindle and motor, in order to avoid any damage to them[Fig. 20][Fig. 21].

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marathon /lutt6oo

9.3 Magner sensor reset mode

Beeps are made 2 seconds after turning on the power switch while pressing both turbo switch @ and forward/reverse selector(Rev.) switch @ at a time, which mean rhe reset to magnet sensor mode . BY turning speed control knob CZ), it is possible to reset the mode to "Hi," "Lo" and "re" modes.

CAUTION

For safety, the above conditions have been already set when shipped and, therefore, do not change them.

| Reset mode | Function | How to reset |
|------------|------------------|--|
| "Hi" | High level reset | Beeps are made to reset the high level by pressing forward/reverse selector (Rev.) switch @ while completely depressing knee control panel ® |
| "Lo" | Low level reset | Beeps are made to reset rhe low level by pressing forward/reverse selector (Rev.) switch @, without depressing knee control panel ® |
| "re" | Recovery | The magnet sensor reset mode is returned to the main mode by pushing the forward/reverse selector (Rev.) switch. |

9.4 Specifications

9.4.1 Control unit

- Model: SDE-S6O/L6O
- Power supply: AC2OO-24OV, 50/60Hz
- Weight: 2.9kg
- Dimension : 94mm(W) x 268mm(D) x 228mm(H)

9.4.2 Motor handpiece

- Model: SDE-BH6OS
- Speed: 1,000-50,000RPM
- Weight: 23Og(not including the motorcord)
- Dimension: L159 x 027
- Cord length: 1.4m

9.4.3 Foot pedal

- Model: SDE-FS6O
- Weight: 384g
- Cord length: 1.Sm

9.4.4 Control unit stand

- Weight: 152g
- Dimension: 122mm(W) x 268mm(D) x 30mm(H)

9.4.5 Motor handpiece stand

- Weight: 68g
- Dimension: 60mm(W) x 110mm(D) x 45mm(H)

9.5 Troubleshooting

| Troubles | Check points | Corrective actions | |
|--------------------------|--|--|--|
| The power LED is not on. | Make sure that the power cord is correctly plugged. | Correctly plug the power cord. | |
| | Check that the fuse is blown. | Replace with a recommended fuse. If the fuse is still blown again, then repair it. | |
| | Make sure that the power switch is properly working. | Repair if the power switch is troubled. | |
| The motor does not run. | "1E" displayed | Check that the motor cord is plugged in proper condition. Make sure in the maintenance mode that the "He" for hall sensor check is in proper condition. | Correctly plug the motor cord. If the same error is continuously displayed, replace the motor cord. |
| | "2E" displayed | Check that the collet chuck is opened. | If opened, close the collet chuck. If the same error is still displayed, then repair it. |
| | "3E" displayed | Check if the unit has been excessively used under the condition of too high load. | Restart running the unit after keeping it stopped for about 10 minutes. If the same error is still displayed, repair it. |
| | "4E" displayed | Make sure that the supply power is AC200-240V, 50/60Hz. | Check if the unit is properly running while the specified power is supplied. If the same error keeps being displayed, repair it. |
| | "5E" displayed | Check if the unit has been excessively used under the condition of too high load. | Restart running the unit after keeping it stopped for about 10 minutes. If the same error is still displayed, repair it. |
| | | Check if indoor temperature is too warm in a work place with the control unit. | If too warm, move the control unit to a location with the indoor temperature of 0-40°C. |
| | | Make sure that the temperature sensor is in good condition. | If troubled, repair the temperature sensor. |
| | "6E" displayed | Check if the collet chuck is opened. | If opened, tighten it by turning the chuck handle. |
| | | Check if the motor handpiece shaft is rotated by fingers. | If not rotated while closed, the motor handpiece spindle is troubled. Then, repair it. |
| | "7E" displayed | It is necessary to resupply the power. | If the same error is still displayed, repair it. |

marathon
/lutt6oo

| | | | |
|--|----------------|--|--|
| | "SE" displayed | Check if the motor keeps running even after turning off the run switch. | Turn off and then on again the power. If the same error is still displayed, repair it. |
| The foot pedal does not work. | | Make sure rhar the foot pedal cord is properly plugged. | Plug the foot pedal cord correctly. |
| | | Check if the auto cruise switch LED is on. | If not turned on, then repair it. |
| | | Make sure in the maintenance mode char the "Fe" for the foot pedal check is appropriate. | If the pedal is defective, repair or replace with new one. |
| The knee control panel does not work properly. | | Make sure in the maintenance mode that the "nc" for the knee control panel is proper. | If the knee control panel is not working properly, reset it in the magnet sensor reset mode. If the same error is still displayed, then repair the control unit. |
| The motor handpiece is overheated while in operation . | | The ball bearin gs are defective. | Repair the motor handpie ce. |
| The motor handpiece is excesisvely vibrat ed and with serious noise . | | Make sure that the collet chuck is free from dirt. | Thoroughly clean the caller chuck after disasembled. |
| | | Check if the specified test bur/ polishing point is used. | For high speed, use the specified test bur/polishing point only. |
| | | Check if the test bur/polishing point is bent. | Replace with new one. |
| | | Make sure that theball beating is in good condition. | Repair the motor handpiece. |
| The test bur/polishing point is released. | | Check if the collet chuck is loose. | If loose, tighten the collet chuck. |