Instructions for use

Product description:

Bifix QM is a radioopaque, light and chemically curing resin-based luting system for durable adhesive fixing of pre-treated metal, ceramic and synthetic pieces. **Bifix QM** is available in three shades: universal, transparent and white-opaque. QuickMix syringes deliver bubble-free, perfectly mixed material that can be directly applied from the syringe. The **Bifix QM** system achieves excellent adhesion with suitable dentine/enamel bonds (**Futurabond DC**) and **Ceramic Bond** for silanization of the restoration.

The **Bifix QM Try-In** pastes are intended to assist the dentist with selecting a shade for luting ceramic restorations with **Bifix QM**.

Indications:

Adhesive luting of inlays, onlays, facettes, veneers, crowns, partial crowns, pins, posts, bridges and adhesive bridges (Maryland bridges).

Method of use:

Please consult the instructions for use for **Futurabond DC** and **Ceramic Bond** for detailed information and application steps for each product.

1. Preparation

Remove the temporary if applicable. Clean, thoroughly rinse and slightly dry the tooth with air.

Check the shade effect of the restoration with ceramic and composite pieces by inserting the definitive restorative after applying the selected shade of **Bifix QM Try-In**. The colouring of the **Bifix QM Try-In** corresponds exactly to the respective **Bifix QM** luting composite shade after curing.

Apply **Bifix QM Try-In** paste on the inside of the piece to check the shade. Carefully insert the piece with light pressure. Do not perform an occlusion check (danger of breaking the restoration).

If multiple restorations must be luted, start in the middle of the arch and insert the pieces one after the other from mesial to distal.

After checking shade effect, remove the pieces and thoroughly clean the **Bifix QM Try-In** paste from both the tooth and the restorative with. Subsequently dry the dentine being careful not to over-dry it.

Note

The work field must be kept clean and dry. A rubber dam should be applied after the shade of the luting composite has been selected. Teeth covered with a rubber dam may dry out which can slightly influence the impression of the shade.

2. Preliminary preparation

Protect dentine in close proximity to the pulp with a suitable lining, e.g. calcium hydroxide and/or glass ionomer cement.

3. Conditioning the restoration

a. Full ceramic restorations
The inside adhesive surfaces of the restorative should be micro-etched with a hydrofluoric acid etchant. Normally, the dental laboratory has already done this.

Subsequent to the try-in with **Bifix QM Try-In**, etch the restoration for 15 s with a phosphoric acid gel, then rinse and dry.

Dispense **Ceramic Bond** (silane) in a mixing well. Apply **Ceramic Bond** to the inside of the restorative with a brush and allow to work for 60 s. Afterwards, dry for 5 s with oil-free air. Do not touch or otherwise contaminate the conditioned surfaces.

b. Metal, composite and zirconium oxide restoratives
Sandblast the surfaces with maximum 1.5 – 2 bar pressure and a suitable agent. Follow the manufacturer's instructions for use for the respective material.

Carefully remove the blasting abrasive residue and dry the adhesive surfaces. Apply **Ceramic Bond** with a brush to the conditioned surfaces, allow 60 s for reaction and then carefully dry with oil-free air. Do not touch the adhesive surfaces afterwards and quickly lute the restoration with **Bifix QM** while observing the following points.

c. Chairside luting of inlays
Clean the bonding surface with alcohol and dry with oil-free air. Do not touch the adhesive surface thereafter – insert and bond the restoration in accordance with the following points.

4. Application of the bonding
Using **Futurabond DC** is recommended for bonding **Bifix QM**. Follow instructions for use for the respective bond.

5. Luting the restorative with **Bifix QM**
Insulate any removable prosthodontic elements before luting. Only mix **Bifix QM** immediately before luting, as described in the following: Due to technical reasons slight differences in the filling level between the syringe chambers can occur prior to the first use. Therefore remove syringe cap and squeeze the material out until the material is delivered evenly from both chambers. Attach a mixing tip type 11. The notches on the syringe and the mixing tip must line up exactly. Turn the mixing tip 90° clockwise to lock the cannula into place. The material is automatically mixed in the mixing tip and can be directly applied onto the prepared contact areas. If needed, the optional intraoral tip type 4 can be placed onto the mixing tip to directly apply the material onto the tooth or prepared root canal.

Working time under reduced light is approximately 2.5 – 3.5 min. Setting time is 3 min from the time of inserting the prosthodontic piece.

Insert the piece and fix it with slight pressure.

Remove excess material with a foam pellet or a disposable brush. Using dental floss is recommended for removing excess in interdental spaces. Leave a slight excess (see below).

Advice for luting Maryland bridges
For luting Maryland bridges with the use of **Futurabond DC** as bonds, the enamel must either be roughened or conditioned with a phosphoric acid etching gel before applying the bond.

6. Luting of fibre-reinforced resin posts (in combination with **Futurabond DC**):

1. Massage the mixed **Futurabond DC** (liquid 1 and 2) into the occlusal surface of the core and root canal for 20 s with an applicator. Disperse solvent with oil-free air for 5 s. Remove any material excess in the canal with an absorbent strip of paper. Do not light-cure the bonding layer.

2. It is recommended to apply **Bifix QM** directly into the root canal using fine intraoral tips type 1. Do not use a lentulo! After preparing the root pin according to the manufacturer's instructions, it can be additionally moistened with **Bifix QM**. Insert the post making sure to leave a slight excess.

3. To fix the post and cure the **Futurabond DC** applied to the occlusal surface, light-cure for at least 40 s.

7. Light-curing
The polymerization time can be controlled through additional light-curing. Light-cure the material at the luting gap with a halogen or LED light for a minimum of 20 s per segment. The light output should be a minimum of 500 mW/cm² on halogen polymerisation devices and 300 mW/cm² on LED devices. Hold the tip of the light rod as close as possible to the surface to be cured. The restorative piece must be fixed in its desired position with a suitable instrument during light-curing.

Advice on oxygen inhibition:
Like all resin-based materials, **Bifix QM** surfaces in contact with oxygen do not cure completely (approximately 50 µm inhibited layer). Therefore, using one of the two following methods is recommended:

- a. Leave a slight excess and remove it during the subsequent finishing; or
 - b. Remove the excess completely and cover the margins of the restorative piece with oxygen-impermeable protective gel.
8. Finishing/Fluoridation
The removal of excess is possible immediately after polymerisation. Remove cured excess with a finishing diamond or flexible polishing disk. Finish approximate areas with finishing and polishing strips. Enamel surfaces which have been excessively etched or damaged during finishing should then be fluoridated.

Warnings and precautionary measures:

Only use suitable bonding materials.

Some enamel/dentine bonding agents are not compatible with dual-curing composites. When in doubt, consult the manufacturer's instructions for use for the respective bond. For best results, use **Futurabond DC**. **Bifix QM** contains Bis-GMA, benzoyl peroxide and amines. **Bifix QM** should not be used in case of known hypersensitivities (allergies) to any of these ingredients. In case of contact with the eyes, immediately rinse with plenty of clean water and consult a doctor.

Irritation to the pulp may occur with insufficient dentine thickness (less than 1 mm). Pulp protection or core build-up is recommended. Residue from thymol, eugenol and/or other preparations containing oil of cloves disturbs the curing process and therefore, should be avoided. Only use oxygen-free temporary cements for temporary luting.

Bifix QM is not suitable for fixing brackets due to its high adhesion values.

Storage:

Store at temperatures between 4 °C – 23 °C. Do not use after the expiry date. Refrigeration is recommended.

Allow **Bifix QM** to reach room temperature before use.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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