# Remin Pro

## Instructions for use

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### Indications :

- After tooth whitening
- After professional tooth cleaning
- For the prevention and control of hypersensitivities
- During orthodontic treatment

### Application:

1. Apply contents of a SingleDose blister of **Remin Pro** to the teeth with a finger or suitable instrument (toothbrush or swab) and distribute.
2. The patient should distribute the remaining **Remin Pro** in his or her mouth with the tongue. **Remin Pro** and the saliva should remain in the mouth for as long as possible (minimum of 3 minutes) for an optimal result.
3. The patient should spit out the remaining amount. Rinsing should be avoided if possible. The patient should wait at least 30 minutes after the treatment before eating or drinking.

### Also for use at home after oral hygiene.

**Storage:**  
Store **Remin Pro** in a cool, dry area protected from direct sunlight (4°C - 23°C). Do not use the material after the expiry date.

### Precautions:

1. **Remin Pro** contains parabene, fluoride and hydroxy apatite (calcium and phosphate). Do not use in cases of known allergies to any of the ingredients.
2. The application of **Remin Pro** is not recommended for patients under 6 years or for home use on children under 12 years.
3. In case of contact with the eyes, immediately flush thoroughly with plenty of water and seek medical attention.
4. Immediately discontinue treatment and seek medical attention if symptoms of angio-oedema occur that can be attributed to the ingredients of **Remin Pro**.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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