VOCO

Registrado X-tra

GB Instructions for use

Registrado X-tra is an A-silicone for bite registration with extreme final hardness that quickly sets. Highly thixotropic properties prevent dripping from the teeth and enable application in thicker layers. The special temperature-controlled setting formula eliminates time pressure during application and has an extremely short setting time in the mouth. Afterwards the registration can be removed without breaking or warping. **Registrado X-tra** can be easily trimmed with a scalpel or rotating instrument and stays dimensionally stable.

Indications:

Registrado X-tra is suitable for bite registration, fixing of face bow registration and is a location material for intraoral pin tracing templates.

Directions:

Use **VOCO** Dispenser type 2 or any other suitable extruder gun.

Due to technical reasons there may be slight differences in the filling quantity of both cartridge chambers. Therefore, remove cartridge cap after placing the cartridge into the dispenser type 2 and squeeze out material until it is delivered uniformly from both chambers.

Residues from other materials (e. g. impression materials) or solutions may disturb the setting process and should be removed prior to application (if needed, by spraying).

Fix mixing tip type 16 and squeeze out material by pressing the lever. The material is completely and automatically mixed in the correct ratio in the mixing tip.

If desired, an intraoral tip may be inserted in the open end of the mixing tip type 16.

In case of occlusal bite registration **Registrado X-tra** is to be applied directly onto the upper or lower dental arch of the patient.

The working time of **Registrado X-tra** is ca. 30 s (from begin of mixing at a room temperature of 23°C). After 40 s time in mouth **Registrado X-tra** can be removed.

Precautions:

Store at temperatures between 4°C - 23°C. Remaining material in the mixing tip will harden. Do not use after expiry date. Avoid contact of the uncured silicone material with clothes. Do not use with condensation curing silicones or eugenol-containing materials. Latex gloves, mouthrinse and retraction solutions can also have a negative influence on the setting process.

Disinfection of the registrations with an aqueous disinfectant is recommended.

Registrado X-tra contains polydimethylsiloxane and platinum catalyst and should not be used in case of known allergies to these ingredients.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VOCO GmbH