VOCO

Curvy

(GB)

Interdental wedges Instructions for use

Indication:

Aid for placing fillings, for applying pressure to matrices and for temporarily separating teeth to shape the interproximal contact during filling therapy.

The longitudinal curvature facilitates adaption to the natural tooth geometry. The inverse "V" form corresponds to the shape of the interproximal space and reduces the pressure of the wedge on the gingiva during placement of the filling. When inserted into the interproximal space, the wedges follow the anatomy of the tooth and ensure a close adaption of the matrix to the tooth and cavity.

The wedges are arranged by colour according to their topographical range of use, their sizes and their clockwise or anti-clockwise curvature:

Light orange / light blue	= Small	for tight interproximal spaces and for children
Medium orange / medium blue	= Medium	for medium interproximal spaces
Dark orange / dark blue	= Large	for wide interproximal spaces
Deep orange / deep blue	= Extra Large	for wide open interproximal spaces

The significance of the different colours arises from their topographical allocation within the oral cavity:

Range of application of orange wedges with clockwise curvature:

In the 1st quadrant, from mesiobuccal or distolingual direction
In the 2nd quadrant, from distobuccal or mesiolingual direction
In the 3rd quadrant, from mesiobuccal or distolingual direction
In the 4th quadrant, from distobuccal or mesiolingual direction

Range of application of blue wedges with anti-clockwise curvature:

In the 1st quadrant, from distobuccal or mesiolingual direction
In the 2nd quadrant, from mesiobuccal or distolingual direction
In the 3rd quadrant, from distobuccal or mesiolingual direction
In the 4th quadrant, from mesiobuccal or distolingual direction

This determination of the range of application necessarily results from the shape of the wedges: These are curved in one direction along the longitudinal axis and have an inversely V-shaped cross section that is modelled on the shape of the interproximal space. **The broad base of the wedge must always rest on the gingiva, while the tip of the "V" points in the direction of the occlusal plane.**

All wedges bear a clearly recognizable dot on their wide end. This marker must always face the occlusal surface, to ensure that the above requirement is fulfilled.

The correct choice of wedge therefore always results from the position of the cavity to be filled:

- If, for example, tooth 46 is to be disto-occlusally filled and the wedge to be placed from buccal direction, then one needs to use an orange wedge in the appropriate size. If the wedge is to be placed in the same location, but from lingual direction, then a blue wedge must be utilised.

- If, for example, tooth 25 is to be mesio-occlusally filled and the wedge to be placed from buccal direction, then one needs to use a blue wedge in the appropriate size. If the wedge has to be placed in the same location, but from palatal direction, then an orange wedge must be chosen.

- If, for example, tooth 17 is to be mesio-occlusally filled and the wedge to be placed from palatal direction, then one needs to use a blue wedge in the appropriate size. If the wedge is to be placed in the same location, but from buccal direction, then one needs to use an orange wedge.

One can thus use either colour for the same filling. The wedge must, however, be placed once from the palatal or lingual side and once from buccal, as the case may be. This is necessary to **strictly comply with the rule** applicable to these wedges that, at all times, **the broad side of the respective wedge, i.e. its sole, has to rest on the gingiva and the marker has to point in the direction of the occlusal surface of the tooth concerned.**

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 BB2060 E1 0211 99 © by VOCO