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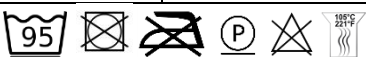
User Manual/ Instruction for use

Anti-Decubitus-Mattress / Decubitus Therapy system with evacuation function

Please read the instructions and the notes on safety carefully before using the system to ensure safe operation!

Product name	EVAQ®-PRO	EVAQ®-PRO VISCO	EVAQ®-PRO II	EVAQ®-PRO-XL
Scope of delivery	Mattress with foam core and PU-TEX Cover with zipper, instruction for use			
Indication	Decubitus Therapy up to stage 2 (EPUAP) middle risk	Decubitus Therapy up to stage 2 (EPUAP) middle risk	Decubitus Therapy up to stage 4 (EPUAP) high risk	Decubitus Therapy up to stage 4 (EPUAP) high risk
Contraindication	none			
Range of application	Decubitus Therapy up to stage 2 (EPUAP) middle risk	Decubitus Therapy up to stage 2 (EPUAP) middle risk	Decubitus Therapy up to stage 4 (EPUAP) high risk	Decubitus Therapy up to stage 4 (EPUAP) high risk
Patient weight	20 kg - 120 kg	20 kg - 120 kg	20 kg - 120 kg	40 kg - 250 kg
Material	PUR- foam COVER: PU-coating/nylon	PUR- foam Cover PU-coating/nylon	PUR- foam Cover PU-coating/nylon	PUR- foam Cover PU-coating/nylon
Sizes	width: 90-100 cm length: 190-220 cm			
Max. patient weight	20kg bis 120 kg	20 kg bis 120 kg	20 kg bis 120 kg	40 kg bis 250 kg
Weight of product (90/200 cm)	11 kg	11 kg	15 kg	20 kg
Medical device class 1 according 93/42 EWG and medical product law (MPG)				
Technical safety/ flame retardancy	DIN EN 597 I + II // BS 7175 sec 3 crib 5			
Biocompatibility	DIN EN ISO 10993-5 ; DIN EN ISO 10993-1			
Notes on safety	Suitable for beds with adjustable lying surface Side bars of the bed must be at least 22cm above surface of the mattress			
guaranty	3 years	3 years	5 years	5 years

Cleaning/ Maintenance/ Further use

Homecare	Clinical use
While using	
Cover disinfection by whipping	Cover disinfection by whipping
	
After changing patient	
Cover disinfection by whipping	Cover disinfection by whipping
Cover disinfection by whipping	Autoclaving
	
No further maintenance necessary	

Instructions for storage and use

1. After receipt, the mattress must be taken out of the cardboard box immediately
2. Put the mattress onto the bed.
3. Please make sure that the arrows on the label („top side“ + mattress name) point upwards.
4. Belts should only be used in emergency cases. Please watch the instruction video on www.funke-medical.de !

Manufacturer according to the MPG (German Devices Act)

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Funke Medical AG