

Instructions for use

Hyper Foam 2

Anti-decubitus mattress / Decubitus therapy system

For high risk up to stage 4 (EPUAP)

Please read the instructions and the notes on safety carefully before using the system to ensure safe operation!

Scope of delivery

- Mattress Hyper Foam 2 with foam core, blue PU-Tex cover with a zipper and welded seams, instructions for use

Indications / Contraindications

- Indications:
 - Prophylaxis against decubitus ulcers with a low, middle and high risk
 - Treatment of decubitus ulcers up to stage 4 (EPUAP)
 - Applicable for pain therapy
- Contraindications:
 - None

Range of application

- The anti-decubitus mattress Hyper Foam 2 is suitable for decubitus prophylaxis and therapy up to stage 4 (EPUAP) and pain therapy.

Permissible patient weight

- The anti-decubitus mattress Hyper Foam 2 is suitable for patients with a body weight from 20 kg to 120 kg.

Technical data



- Foam mattress:

HR 4520	density: 45 kg/m ³	compressive strength: 2,0 kPa	HR 5030	density: 50 kg/m ³	compressive strength: 3,0 kPa	T 3545	density: 35 kg/m ³	compressive strength: 4,5 kPa
---------	----------------------------------	----------------------------------	---------	----------------------------------	----------------------------------	--------	----------------------------------	----------------------------------

- Cover material: polyurethane + polyester
- Weight of product: approx. 10 kg

Dimensions

- 86/186/15 cm
- 86/196/15 cm
- 105/196/15 cm

<u>Cleaning and maintenance</u>		Homecare	Clinical use
While using			
✓	cover disinfection by wiping (no alcohol and no chlorine)	✓	cover disinfection by wiping (no alcohol and no chlorine)
✓			
After changing patient			
✓	cover disinfection by wiping (no alcohol and no chlorine)	✓	cover disinfection by wiping (no alcohol and no chlorine)
✓	autoclaving	✓	autoclaving
✓			

- The Hyper Foam 2 is a medical device according to RL 93/42, RL 2007/47 and to German Devices Act.
- Maintenance is not necessary.

Technical security / Flame retardancy

- Flame retardancy testing according to BS 7175 Crib 5, DIN EN 597 Part 1 + 2

Toxicological and biocompatible tests

- DIN EN ISO 10993-5 ; DIN EN ISO 10993-1

Instructions for storage and use:

1. After receipt, the mattress must be taken out of the cardboard box immediately.
2. Put the mattress onto the bed.
3. Please make sure that the arrows on the label ("top side"+ mattress name) point upwards.

Notes on safety

- Suitable for beds with adjustable lying area
- The side bars of the bed must be at least 22 cm above the upper edge of the mattress.

Product name

- Funke Medical Hyper Foam 2

Manufacturer according to the MPG (German Medical Devices Act)

Funke Medical AG
 Ährenfeld 10
 46348 Raesfeld
 Tel.: +49 - 2865-6648
 Fax: +49 - 2865-8133
info@funke-medical.de
www.funke-medical.de
 - GERMANY -