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**User manual / Instruction for use**  
**Positioning Aids**  
**POSIMED®**

Please read the instructions and notes on safety carefully before using the systems to ensure safe operation!

**Scope of delivery**

- Positioning Aid POSIMED® with blue PU cover and welded seams, user manual

**Indications / Contraindications**

- Indications: Therapy and prophylaxis of pressure ulcers and positioning of pain patients.
- Contraindication: None

**Range of application**

- Positioning aids in various shapes and sizes to ensure an optimal positioning for the decubitus prophylaxis and therapy support.

**Patient weight**

- Suitable for all patient weights

**Technical data**

- Positioning aid filled with polypropylene microbeads

Neopolen	Density 13 – 17 kg/m <sup>3</sup>
----------	-----------------------------------

**Dimensions**

- Various shapes and sizes

**Weight**

- 0,2 kg to 1 kg

Cleaning and maintenance	
Homecare	Clinical use
<b>While using</b>	
Cover disinfection by wiping	Cover disinfection by wiping
<b>After changing the patient</b>	
Cover disinfection by wiping	Cover disinfection by wiping
Cover disinfection by wiping	Autoclaving

**Technical safety / Flame retardancy**

- DIN EN 597 I + II Flame retardancy

**Toxicological Test / Biocompatibility**

- DIN EN ISO 10993-5 ; DIN EN ISO 10993-1

**Preparing the system for use**

- Remove the positioning aid from the package!

⇒ The system is ready for use now.

**Warnings**

- None

**Guaranty / Warranty**

- The statutory guaranty is 6 months and the statutory warranty is 2 years. Extended guaranty conditions can be found in the product descriptions.

**Product name**

- Positioning aid POSIMED®

**Manufacturer according to the MPG (German Medical Devices Act)**

Funke-Medical AG  
 Ährenfeld 10  
 46348 Raesfeld  
 Tel.: (+49) 2865/266 93 50  
 Fax: (+49) 2865/8133  
[info@funke-medical.de](mailto:info@funke-medical.de)  
[www.funke-medical.de](http://www.funke-medical.de)