

# VIRACLEAN®

HOSPITAL GRADE DISINFECTANT  
AUST L 69000

## ABOUT VIRACLEAN®:

VIRACLEAN® is a major development in advanced cleaning and disinfecting technology from Whiteley Medical. It is the result of years of intensive research and development. A considerable investment in both time and money has been made in the interests of improving the safety and efficacy of hospital grade disinfectants. VIRACLEAN® can be used for both cleaning and disinfecting, demonstrating beneficial versatility for healthcare and associated environs.

VIRACLEAN® has been proven to kill a broad range of bacteria and viruses including *Enterococcus faecalis* (VRE), *Staphylococcus aureus* (MRSA), Hepatitis B virus, Herpes Simplex virus, Influenza virus, and kills Coronaviruses including SARS-CoV-2 (COVID-19).

## MICROBIOLOGICALLY TESTED:

The VIRACLEAN® label lists that it is proven to kill:

- Acinetobacter
- Candida albicans
- Coronaviruses including SARS- CoV-2 (COVID-19)
- Enterococcus Faecalis (VRE)
- Escherichia coli (E coli)
- Hepatitis B Group virus
- Herpes Simplex virus
- Influenza virus
- Klebsiella pneumoniae (CPE/CRE)
- Proteus vulgaris
- Pseudomonas aeruginosa
- Salmonella choleraesuis
- Staphylococcus aureus (MRSA or Golden Staph)

*\*Independent test reports available on request.*

## PRODUCT DESCRIPTION:

VIRACLEAN® is a pink liquid with a lemon fragrance. It is easy to use and is conveniently packaged in squeeze or spray bottles, 5 litre containers, and 15 litre containers. This product is not intended for use on therapeutic devices. INSTRUMAX® PINK is available as an instrument disinfectant low level for use in disinfecting non critical medical devices.

**PRODUCT FEATURES:**

**VIRACLEAN®:**

- Is non flammable
- Is suitable for use on healthcare surfaces
- Does not contain chlorine or oxidising biocides
- Will not cause staining on fabrics or carpets
- Is pH neutral
- Non corrosive to metals
- Has a mild lemon fragrance

**CHEMICAL SPECIFICATIONS:**

Surfactant Blend	Blended
Colour	Light Pink
pH	Neutral (7)
Specific Gravity	0.99
Corrosion Inhibitor	Not necessary
Foaming Properties	Slight
Dangerous Goods Rating	Not applicable
Shelf Life	18 months
Safety Equipment	Use of gloves and eye protection recommended

**VERSATILE:**

VIRACLEAN® is formulated to replace most cleaning products and hospital grade disinfectants. VIRACLEAN® can be used for cleaning and disinfecting.

**HELPS CONTROL INFECTION:**

The use of gloves is recommended whenever using disinfectants.

To remove organic soiling (blood, sweat, sputum etc) from a surface, it is recommended to apply Viraclean® liberally around the soiled area and work towards the centre. Soak up using fresh, disposable cloth.

Once clean, to disinfect the surface, apply undiluted and leave to dry for 10 minutes at room temperature. If required, soak up using fresh, disposable towelling. Used towelling should then be disposed to a suitable receptacle.

**EDUCATIONAL VIDEO AVAILABLE:**

An Educational video is available on 'How to clean and disinfect blood spills using VIRACLEAN®'. To watch the educational video visit [www.whiteley.com.au/education](http://www.whiteley.com.au/education)

**PRODUCT USE:**

<b>Nursery:</b>	Cleaning cribs, cots, change tables.
<b>CSSD:</b>	Bench tops, stainless steel surfaces, sinks, trolleys, tap fittings.
<b>Toilet Shower:</b>	Sinks, tile surfaces, and all hard surfaces.
<b>Intensive Care Units:</b>	Cleaning and disinfection of all hard surfaces.
<b>Manual Cleaning of Theatre Equipment:</b> <i>(excludes medical devices)</i>	VIRACLEAN® can be used to clean and disinfect equipment and cables and will not adversely affect surfaces.
<b>Operating Theatres:</b>	Cleaning and disinfecting of tables in theatre.
<b>Blood Spills:</b> <i>(excludes medical devices)</i>	VIRACLEAN® is ideal for cleaning and disinfecting blood spills on all hard surfaces.
<b>Telephones:</b>	VIRACLEAN® is ideal for cleaning and disinfecting phones where large numbers of bacteria are found.
<b>Dental &amp; GP Surgeries:</b> <i>(excludes medical devices)</i>	VIRACLEAN® can be used in Dental & GP Surgeries for cleaning and disinfecting surfaces including stools, tables, benches and light fittings.
<b>Laboratories &amp; Pathology Clinics:</b>	VIRACLEAN® is excellent for wiping down benches, disinfection of white rooms, cleaning and disinfection of fume cupboards, etc.

**SAFETY DATA:** Information is listed in the Safety Data Sheet.

**PACK SIZE:** 12 x 500mL Squeeze, 12 x 500mL Surface Spray, 2 x 5 Litre, 15 Litre.

**ENVIRONMENTAL STATEMENT:**

Whiteley Corporation manufactures products that comply with Australian and International environmental policy. Whiteley Corporation aims to ensure that every product is 100% biodegradable, whilst achieving optimal performance and benefit to customers. All Whiteley Corporation products are supplied in 100% recyclable containers. Whiteley Corporation and its staff are fully committed to ensuring that our Australian environment and those international environs in which our products are used are enhanced by the use of technology, and that these environments are preserved for the enjoyment of future generations.

© This document remains property of Whiteley Corporation Pty Ltd.

Alterations are not permitted without prior written authorisation from Whiteley Corporation Pty Ltd.